


TPN MODE INFUSION

Access TPN Mode Infusion

User may choose the TPN auto ramp infusion mode from CONFIGURATION screen after entering a new infusion

4. Turn on the pump and select NEW INFUSION.
5. Use Up/Down Arrow keys to highlight **4. TPN MODE** and press the Select soft-key.
6. The pump will display the Programming screen of the TPN infusion.

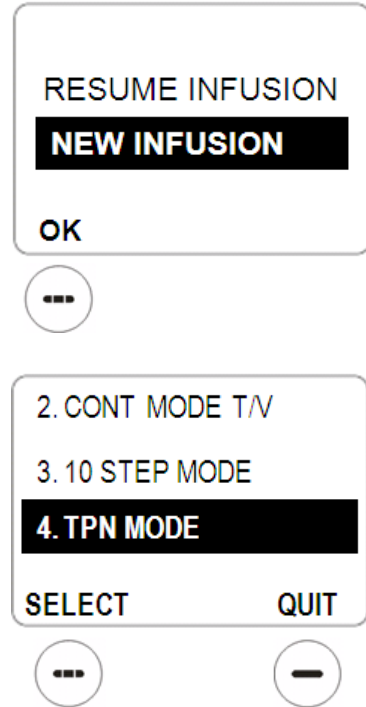


TPN MODE INFUSION (Continued)

Programming TPN Mode Infusion

TPN infusion programming screen contains the following display areas:

1. **Status Bar:** The Status Bar displays current operating mode; current pump state; and volume infused.
 - **TPN:** TPN mode operation.
 - **SET:** Pump programming state.
 - **VINF:** Volume Infused for the current infusion.
2. **Infusion Parameters:** TPN total VTBI and Total TIME of the infusion.
 - **TIME:** Total TIME of the infusion.
 - **VTBI:** Total Volume To Be Infused.
3. **Soft Keys:** Allow user to access pump configuration and TPN ramp programming screen.
 - **CONFIG:** Access pump configuration.
 - **RAMP:** Access TPN Ramp Up and Ramp Down Time parameter programming.
 - **Up/Down Arrows:** Highlight the selected parameter item.

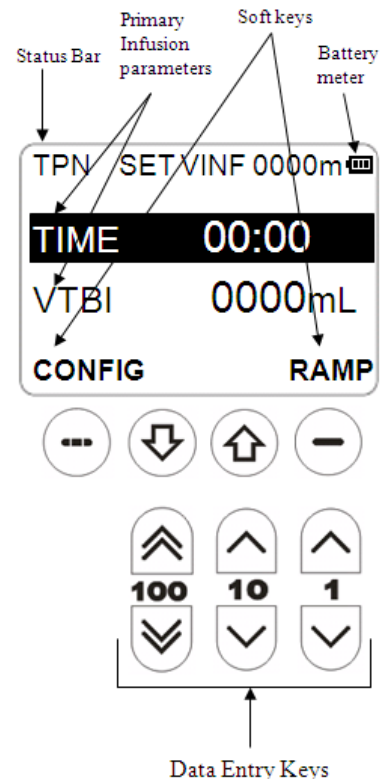

The TPN infusion parameters may be programmed by using the UP/DOWN Arrow keys and Data Entry keys.

Access TPN Ramp Time Parameter Programming

From TPN programming screen, press RAMP Soft-Key to access TPN Ramp Parameter programming screen.

Programming TPN Ramp Time Parameters

The TPN Ramp Time parameters may be programmed by using the UP/DOWN Arrow keys and Data Entry keys.

⚠ WARNING: Do not over-program VTBI. Program the actual amount of the fluid in the IV bag.

